

QUICK START GUIDE


Smart Connect



THE SMART CONNECT AT A GLANCE

The Smart Connect can be used immediately, without programming in Connexx. The various functions will then apply to the universal program.

- For setup/pairing, follow the instructions on page 3.
- For hearing programs with individual settings in Connexx, follow the instructions on page 4.

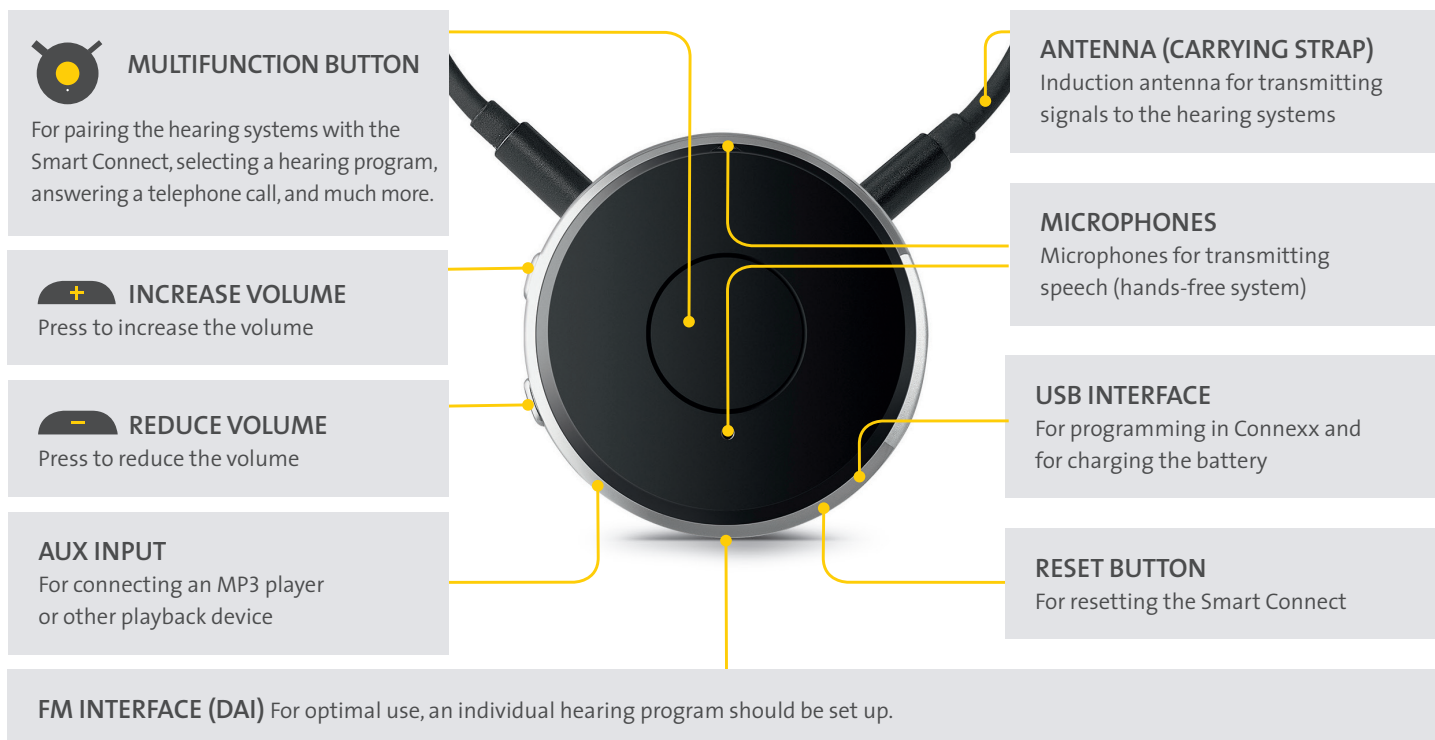


CONNECTION OPTIONS



SMART CONNECT APP



The Smart Connect app makes the hearing systems even more intuitive to use: Change hearing programs, adjust volume, battery status, and much more.

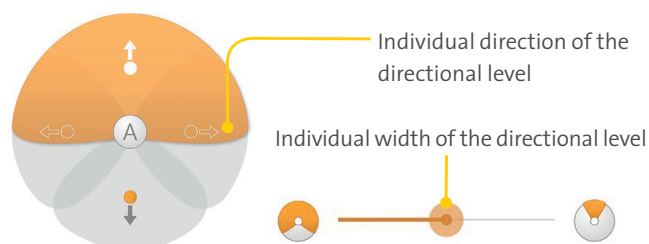

Scan the QR code or look for "Connexx Smart Connect" or "Audio Service" in the stores.



LED DISPLAY/BLUETOOTH PAIRING

BEFORE SETUP, ENSURE THAT ...-

- ! You have your G3 hearing systems to hand.
- ! G3 hearing systems are programmed and operational.
- ! The Smart Connect is charged.
- ! The carrying strap (neck loop) on the Smart Connect is connected.

We recommend connecting the Smart Connect to Connexx before every startup so as to run any available firmware update.

See Smart Connect user guide, page



CHARGING

- Steady orange light: The Smart Connect is charging.
- Steady green light: The Smart Connect is fully charged.

8

DURING USE

- Flashing orange after pressing any button: The battery is low.

Press simultaneously to see whether the Smart Connect is already switched on. The green LED will flash if the Smart Connect is on. Otherwise switch it on by holding down for about 3 seconds.

9

1. FIRST SETUP/PAIRING

1. Switch on the Smart Connect by pressing and holding down for about 3 seconds.
2. Now the Smart Connect automatically searches for hearing systems and Bluetooth devices:
3. Switch on the hearing systems. If pairing is successful, the green LED lights up and then goes out again
4. Until the first Bluetooth device (not our transmitters) is connected, the Smart Connect remains in search mode.
5. If you do not want to connect a Bluetooth device right now, press to discontinue search mode.

11

–

12

2. PAIRING BLUETOOTH DEVICES (hearing systems must be paired)

1. Hold down at the same time until the blue LED flashes: the Smart Connect searches for BT devices.
2. After the connection confirmation on your Bluetooth terminal device (if necessary, enter the PIN "0000"), the blue LED lights up for 5 seconds and then goes out again → connection successfully established:

19

–

21

3. PAIRING OUR TRANSMITTER (hearing systems must be paired)

1. Switch on the transmitter.
2. Hold down at the same time until the blue and green LEDs flash alternately:
3. After successful pairing, the blue LED briefly flashes before changing into a steady light: → Connection successfully established.
4. If the streaming does not begin automatically, press to discontinue the search process.
5. Then press twice quickly: → Connection successfully established.

22

–

23

4. CANCEL PAIRINGS (our transmitters can only be removed using the Smart Connect app!)

1. To unpair all Bluetooth devices, hold down at the same time for approximately 10 seconds.
2. Successful unpairing is indicated by
3. Then the Smart Connect automatically searches for hearing systems and Bluetooth devices once again:


32

(see 1.2)


SETUP IN CONNEXX/ FIRMWARE UPDATES

Configuration


- Instrument
- System Sounds
- Remote Controls



To use the Smart Connect as a remote control, you need to enable it in Connexx. To do this, proceed as follows:

- Under the menu item **Configuration** find the subitem **Remote Controls**. Here you will find all of the configurable remote controls for the connected hearing system.
- Select the Smart Connect remote control with one mouse click. To connect the Smart Connect to the hearing systems, press 

The following icons will change as shown to let you know that you have successfully enabled the remote control:

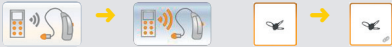

! If a software update is available, it must be carried out promptly. Regularly check whether new updates are available.

Program Handling

1	Universal	
2	Bluetooth Phone	→ To make wireless calls via smartphone/Bluetooth telephone
3	Streaming Audio	→ To transmit audio signals via Bluetooth (e.g. smartphone music app)
4	Smart Connect FM	→ To use FM-capable devices
5	Telecoil (T)	→ To use induction loop systems or telephones
6	Off	

Next, the hearing programs for Smart Connect can be set up.

- Click **Program Handling** in the menu bar.

Below you will find the various hearing programs which are adapted to the customer's needs, depending on the Smart Connect area of application.

To apply the settings for the Smart Connect, after programming in Connexx, you must re-connect the Smart Connect and the hearing systems by switching both systems on/off.